



# Closed-Tray Technique



**STEP 1:** Use appropriate instrument to remove healing abutment. Start with posterior implant, working forward. Remove one healing abutment

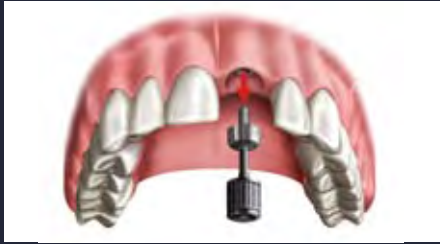

**STEP 2:** Place the impression post (impression coping) onto the implant, ensure the two pieces are fully engaged, and hand tighten the screw.

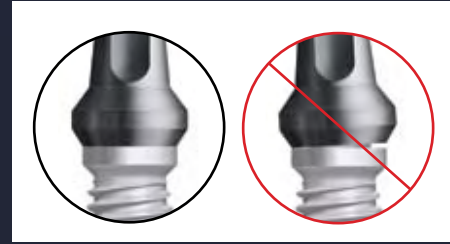

**STEP 3:** Take a PA to verify all parts are seated. When working with multiple implants, place all impression posts in one arch before taking PA.



**STEP 4:** Inject your preferred PVS syringe material around each impression post, fill the tray with heavy body and record impression.



**STEP 5:** Using the recommended instrument, remove the impression posts from the patient's mouth and replace the healing abutment.



**STEP 6:** Place the impression posts into the corresponding location in the impression and send the case to Glidewell Laboratories.